


NEURO MASSAGE THERAPY CENTER

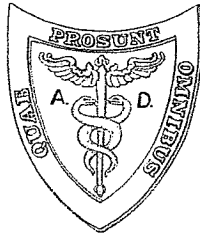


WWW.NMTINSTITUTE.COM

A.D.Exersize Inc. 225-665-7982


POSTURE CORRECTION MAGNETIC STILL POINT SYSTEM

This unique system combines the effects of:

- Magnetic Induction (muscle relaxation / pain relief)
- Cranial System Still Point (nervous system reset)
- Sacral Decompression (low back pain relief)
- Thoracic Decompression (neck and shoulder pain relief)
- Cervical Decompression (headache relief)

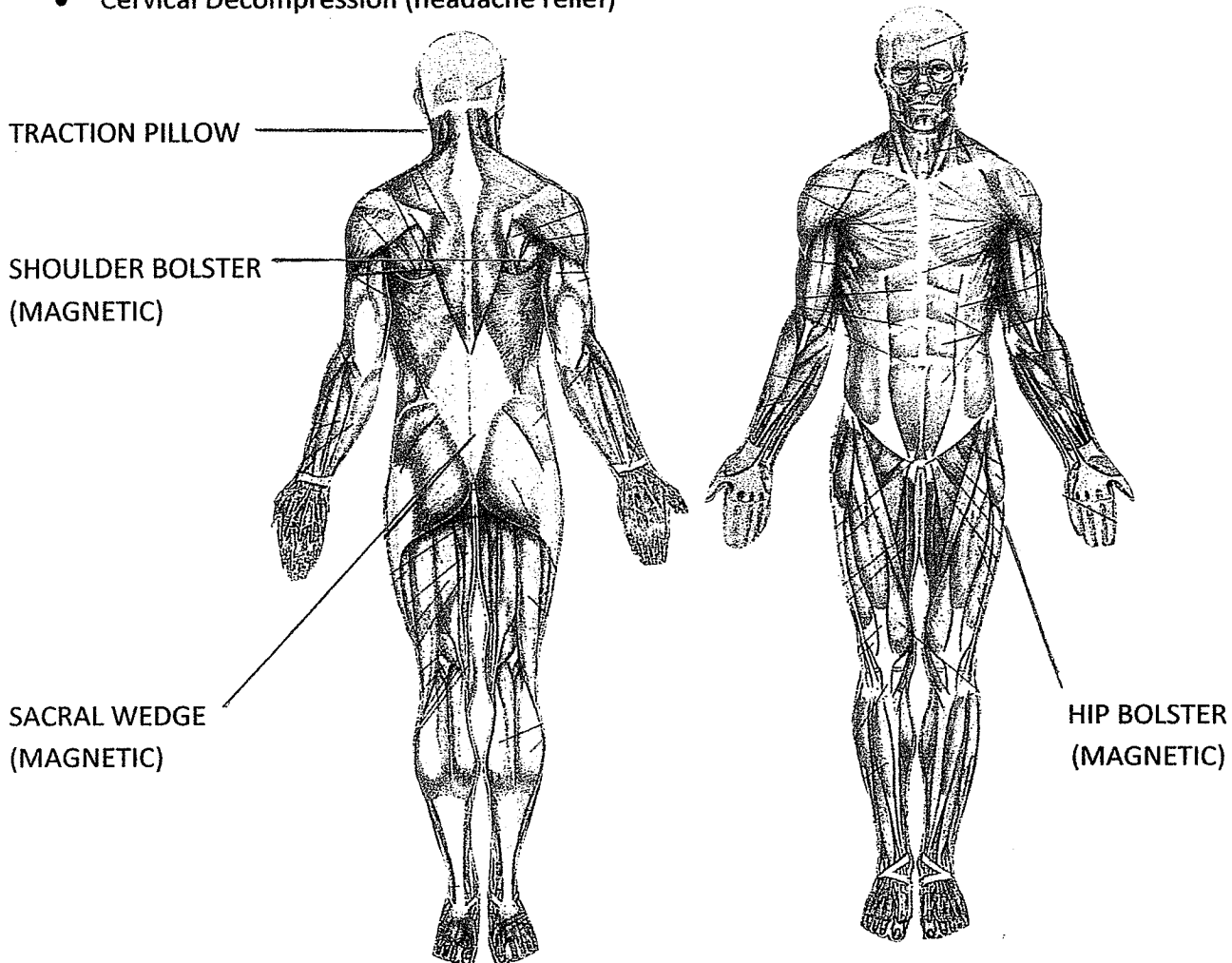

NEURO MASSAGE THERAPY CENTER

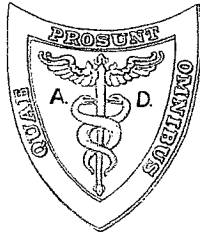

WWW.NMTINSTITUTE.COM

A.D.Exersize Inc. 225-665-7982

Directions:

In the Supine position:

Place the sacral wedge under the sacrum

Place the shoulder bolster with the lower tip of the shoulders in the center of the shoulder bolster.

Place the traction pillow folded and opening up under the neck.

In the Prone position:

Place the hip bolster in the center of the (ASIS) hip bones.

- DO NOT USE IF YOU HAVE AN ELECTRICAL IMPLANT?
- DO NOT USE ON CHILDREN.
- DO NOT USE ON PREGNANT WOMEN.
- LIMIT USE TO 30 MINUTES PER FOUR HOUR INTERVALS.
- CONSULT YOUR PHYSICIAN IF YOU HAVE ANY CONCERNS IN THE USE OF THIS ITEM.
- KEEP ONE FOOT AWAY FROM ANY CREDIT CARDS, CELL PHONES, COMPUTERS, AUDIO & VIDEO TAPES OR MAGNETIC SENSITIVE ELECTRONIC EQUIPMENT.

For any information about this product please contact A.D. Exersize Inc. 33336 Duff Rd, Walker La 70785.
225-665-7982 WWW.NMTINSTITUTE.COM